

Goldshield Recommended Application Matrix.			
LOCATION	FREQUENCY	DISPENSING MET	PRODUCT
All Locations	Daily Use	Trigger Spray	Goldshield 90 TRU
All Locations	Daily Use	wipe	Goldshield 2 wet wipes
Blood cuffs	once a week	Trigger Spray	GS 75 RTU
lead wires	once every 2 weeks	Trigger Spray	GS 75 RTU
Bed rails	Once every 30 days	Trigger Spray	GS 75 RTU
Door knobs	once a week	Trigger Spray	GS 75 RTU
Remote controls	Once every 30 days	Trigger Spray	GS 75 RTU
Phone cradles	Once every 30 days	Trigger Spray	GS 75 RTU
Tray Tables	Once every 30 days	Trigger Spray	GS 75 RTU
Bed table	Once every 30 days	Trigger Spray	GS 75 RTU
Patient chair	Once every 30 60 days	Trigger Spray	GS 75 RTU
Finger soap dispenser	Once every 30 days	Trigger Spray	GS 75 RTU
Elevator buttons	Once a week	Trigger Spray	GS 75 RTU
Elevator hand rails	Once every two weeks	Trigger Spray	GS 75 RTU
wheel chairs and trolleys	Once every 30 60 days	Steam	GS 5 diluted
Waste recepticals	Once every 30 days	Steam or Trigger	GS 5 diluted
Toilets	Once every 14 30 day	Steam/power sprayer	GS 5 diluted
Bowls sinks, metal pipes Once every 14 30 day		Steam/power sprayer	GS 5 diluted
Faucets	Once a week	Trigger Spray	GS 75 RTU
Gurneys	Once every 14 30 day	Steam/power sprayer	GS 5 diluted
General equipment	Once every 30 60 days	Steam/power sprayer	GS 5 diluted
Key boards	Once every 14 30 day	Steam/power sprayer	GS 5 diluted
Vinyl Mattress pads	Once every 30 60 days	Steam/power sprayer	GS 5 diluted
Room storage area	Once every 30 60 days	Steam/power sprayer	GS 5 diluted
ICU Floors and under bed	Once every 30 60 days	Steam/power sprayer	GS 5 diluted
Privacy Curtains	once every 2 weeks	Steam or Trigger	GS 5 diluted
Walls	Once every 90 days	Steam/power sprayer	GS 5 diluted
window wells	Once every 90 days	Steam or Trigger	GS 5 diluted
Other items			
Stethoscopes	Once a week	GS 75 RTU	
Out patient and reception	Once every 30 60 days	GS 75 RTU	
Hall way floors	Once every 30 60 days	Steam/power sprayer	GS 5 diluted

hall ways floors	Once every 30 60 days	steam/power sprayer	GS 5 diluted
EMS	Once every 30 60 days	Steam/power sprayer	GS 5 diluted
HAND SANITIZER	Twice a day	foam dispense	GS 24

STRIKEHIS.COM

<b>COMMENTS</b>
Daily broad spectrum Disinfectant Available March '18
Daily broad spectrum disinfectant Available April '18
reapply during patient turnover
reapply during patient turnover
reapply during patient turnover
If there is high traffic and you assay above 30 re apply more frequently
reapply during patient turnover
reapply during patient turnover
reapply during patient turnover
use with cloth or microfiber cloth
use with cloth or microfiber cloth
use with cloth or microfiber cloth
These get lots of traffic be sure to permit to dry before wiping
use with cloth or microfiber cloth
Be sure to under arm rests and seats all joints.
Be sure to apply under the lid tops where hand contact is frequent
This adds protection at low pH around 3.5 diluted, you should use you normal daily cleaning procedure but you could cut back
In the presence of organic matter steam or liwuid water and detergent should be used disinfectants will deactivate
Takes longer to dry on steal so you may want to apply and wiping after treaing other surfaces
Be sure to treat under handle bars
Apply to base poles, wheels
Apply using less moisture and more heat vapor settings
If there is a blood release, clean first before applying, if blood releases after application just wipe surface clea no need to
There is an accessory that will power spray/fog into the cabinets and give great coverage
Set for more moisture
If protocol is to launder, treat the 12 inches on the edges where people touch frequently depending on material you may be able
AN attachment that could spray walls, focus near patient areas
spray, permit to dry and wipe
Chairs, tables, reception desk tops and bottoms you can spray carpets or apply by steam

Be sure to do ceiling, key boards and floors under gurney, carrying medical cases.
if hand washing is more readily available you may have to add three times per day.

STRIKEHS.COM